

ZAPPY™

IPA WIPES

70% v/v Isopropyl Alcohol



Zappy Alcohol Wipes are a single use, spun-lace wipe saturated with 70% Isopropyl Alcohol solution. Zappy Alcohol Wipes are a convenient and cost effective solution for disinfection of hard surfaces.

Hard Surface Disinfection



Tested against the following EN Standards:

• EN 1276 • EN 13727 • EN 14348 • EN 14476 • EN 14561

Active Ingredient:

Isopropyl Alcohol 70% v/v and deionised water


Product Performance:

Zappy Alcohol Wipes rapidly kill harmful germs, making them ideal for disinfection of many hard surfaces in both clinical and commercial applications. With 70% v/v Isopropyl alcohol Zappy Alcohol Wipes effectively kill 99.99% of harmful germs. These wipes are made from our signature spun-lace paper, which is tear-resistant and soft to touch. By utilising this spun-lace paper, Zappy Alcohol Wipes retain alcohol saturation making them an effective and efficient solution for hard surface disinfection. Zappy Alcohol Wipes are optimal for disinfecting hard surfaces in many environments such as hospitals, nursing homes, child-care, veterinary clinics, dental surgeries, laboratories and other commercial premises.


Instructions Of Use:



Step 1: Wipe surface in an S shape. Let the surface air dry

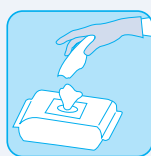

Step 2: Use a new wipe for a new surface

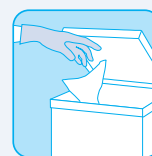

Step 3: Discard used wipes in a waste bin

Precautions:

For hard surface disinfection only, after application allow the surface to dry before use. Avoid contact with eyes and any mucous membranes. If solution comes in contact with the eyes, wash out promptly and thoroughly with fresh water. Skin irritation is uncommon but should a reaction occur discontinue use immediately. Do not use for skin disinfection.

Additional Surface Precautions:

Isopropyl alcohol can damage some forms of plastic, rubber, and lacquered surfaces. Test a small portion of the surface for any damage prior to use.

Shelf Life:

The shelf life of Zappy IPA Wipes is 2 years from date of manufacture printed on the wrapper.

Product Storage:

Zappy IPA Wipes should be stored in a cool, dry, well ventilation location below 30°C. Do not place in direct sunlight or close to any heat source.

Finished Goods Inspection & Testing
QA Testing & Analysis
Total Bacterial Count (TBC),
Total Moulds & Yeast Count (TMYC) – 4 days
Limit: <10
PSB/SGS Confirmatory Test
Monthly Microbial Testing
Antimicrobial Efficacy Testing

Product Specification

Product Code: WTF-FS6777-0003
Product Description: Zappy Alcohol
HS Code: 481820007
Wrapper Material: MCPP/OPP
Finish Product Size: 60 x 135 mm
Paper Material: Spun-lace 40gsm (Nonwoven)
Paper Size: 200mm x 195mm
Industrial 1s Packing: 100 sheets x 20 bags (Total 2000 pieces per carton)
Carton Barcode: 8887245223614
Carton Size: 456 x 386 x 372 mm
Carton Weight: 14.5 kg

