


ZAPPY™

Isopropyl Alcohol 70% v/v Hospital Grade Disinfectant Wipes

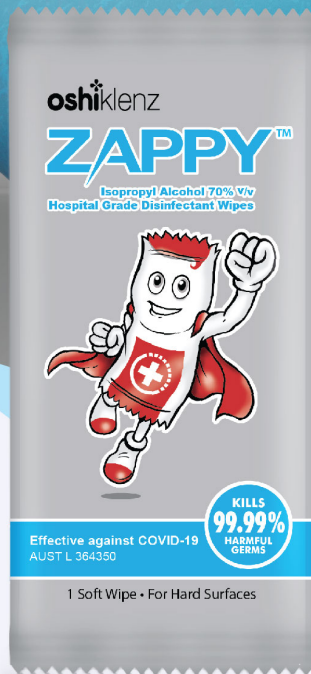

Zappy Alcohol Wipes are a single use, spun-lace wipe saturated with 70% Isopropyl Alcohol solution. Zappy Alcohol Wipes are a convenient and cost effective solution for disinfection of hard surfaces.

Hard Surface Disinfection



Effective against COVID-19

TGA APPROVED - AUST L: 364350

Active Ingredient:

Isopropyl Alcohol 70% v/v and deionised water


Product Performance:

Zappy Alcohol Wipes rapidly kill harmful germs including COVID-19, making them ideal for disinfection of many hard surfaces in both clinical and commercial applications. With 70% v/v Isopropyl alcohol Zappy Alcohol Wipes effectively kill 99.99% of harmful germs. These wipes are made from our signature spun-lace paper, which is tear-resistant and soft to touch. By utilising this spun-lace paper, Zappy Alcohol Wipes retain alcohol saturation making them an effective and efficient solution for hard surface disinfection. Zappy Alcohol Wipes are optimal for disinfecting hard surfaces in many environments such as hospitals, nursing homes, child-care, veterinary clinics, dental surgeries, laboratories and other commercial premises.


Instructions Of Use:



Step 1: Wipe surface in an S shape. Let the surface air dry

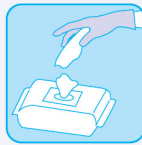

Step 2: Use a new wipe for a new surface

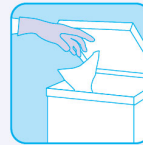

Step 3: Discard used wipes in a waste bin

Precautions:

For hard surface disinfection only. Clean surfaces prior to disinfection, after application allow the surface to dry before use. Avoid contact with eyes and any mucous membranes. If solution comes in contact with the eyes, wash out promptly and thoroughly with fresh water. Skin irritation is uncommon but should a reaction occur discontinue use immediately and see a doctor.

Additional Surface Precautions:

Isopropyl alcohol can damage some forms of plastic, rubber, and lacquered surfaces. Test a small portion of the surface for any damage prior to use.

Shelf Life:

The shelf life of Zappy IPA Wipes is 2 years from date of manufacture printed on the wrapper.

Product Storage:

Zappy IPA Wipes should be stored in a cool, dry, well ventilation location below 30°C. Do not place in direct sunlight or close to any heat source.

Product Specification

Product Code: WTF-FS6777-0008

Product Description: WTE Oshiklenz Zappy Alcohol 1S 2008835

HS Code: 481820007

Wrapper Material: MCP/OPP

Finish Product Size: Foil 60x135

Paper Type: SP40

Paper Size: 200mm x 195mm

Packing/Ctn: 100s x 20pkts

Carton Size External: L456 x W386 x H372mm

Gross Weight/Ctn: 13.7kgs +/-5%

